

### **Contents Software update 220203 for VITA vPad excellence as of serial number 2820180275**

- Orthographic corrections and design changes
- Fast cooling only possible with open firing chamber
- Confirmation signal when a keypad is pressed
- During a running firing other programs can be addressed and changed
- Temperature standby can be activated in the firing curve view
- Liftcontrol with one key
- New users and desktops can be created in the file manager
- Firing programs can be displayed in the file manager
- Acknowledgement of receipt for sent "Online process data"
- "+" icon on the desktop for creating a new firing program and desktop
- "+" icon in the professional view for adding a user profile
- File manager moved from the top bar to settings
- Total number of desktops and current desktop number are displayed on the desktop
- Added standby button and furnace identification button in multi furnace view
- VITA vPad excellence instructions in Bulgarian, Czech, Polish and Russian languages added
- ZYRCOMAT 6100 MS manual added
- Host for online process data has been changed, so that process data can be sent directly to VITA
- Night mode can now be changed during the firing process
- Online update updated with new encryption
- Programs and processing instructions for VITA AMBRIA and VITA SUPRINITY PC updated
- Programs for CEREC Tessera and CEREC MTL Block added
- Glaze firing assistant for veneering materials added
- Firing program parameters can be displayed in the file manager
- SMART-VIEW added as selectable function
- SMART.FIRE mode: up to 4 SMART.FIRE and ZYRCOMAT 6100 MS units can be operated on vPad excellence
- SMART.FIRE mode: Vacuum query with SUPRINITY and IPS e.max CAD

### **Contents Software update 220203 for VITA vPad comfort as of serial number 3020180051**

- Orthographic corrections and design changes
- VITA vPad excellence instructions in Bulgarian, Czech, Polish and Russian languages added
- ZYRCOMAT 6100 MS manual added
- Night mode can now be changed during the firing process
- Programs and processing instructions for VITA AMBRIA and VITA SUPRINITY PC updated
- Programs for and CEREC MTL Block added

### **Contents Software update 220203 for VITA ZYRCOMAT 6100 MS:**

- High-performance power supply is switched off with a delay after the end of the fire to ensure greater operating stability
- Fans are switched off when the furnace is switched off via the vPad